



Instruction for Use EN

Aegis Hand Shower Filter



Description

The filter of the T-safe Pro Water Filter range is based on hollow fiber membrane technology and is specifically designed to remove bacteria such as, but not limited to, *Legionella pneumophila* from tap water in order to prevent infections in commercial settings.

T-safe Aegis Hand Shower Filter is equipped with a male ½” thread. The filters are supplied with a water flow regulator, which protects the filters against sudden water pressure peaks.


T-safe Aegis Hand Shower Filter will effectively retain bacteria for a maximum period of up to 92 days, within the limitations of use as described herein.

This IFU covers the following products

Item	Code
Aegis Hand Shower Filter, 1-function	02-803400
Aegis Hand Shower Filter, 3-function	02-803403
Replacement Filter	02-000300
Waterway Cartridge	02-000122

Installation


1

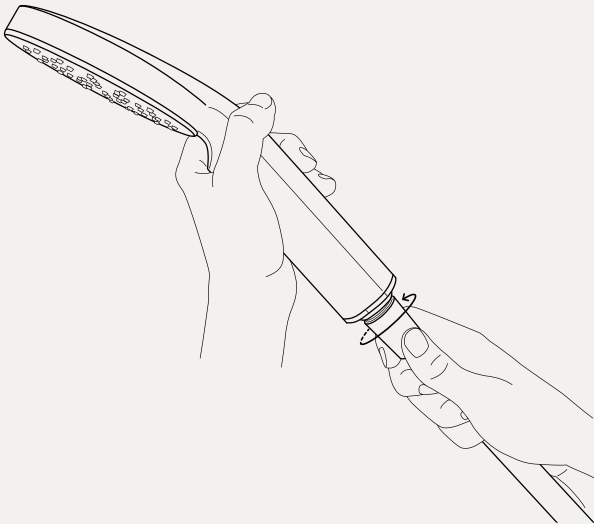

Hand-tighten the shower filter to the hose.

The gasket is integrated into the product to ensure it is replaced with each filter change so no leaking occurs.

2

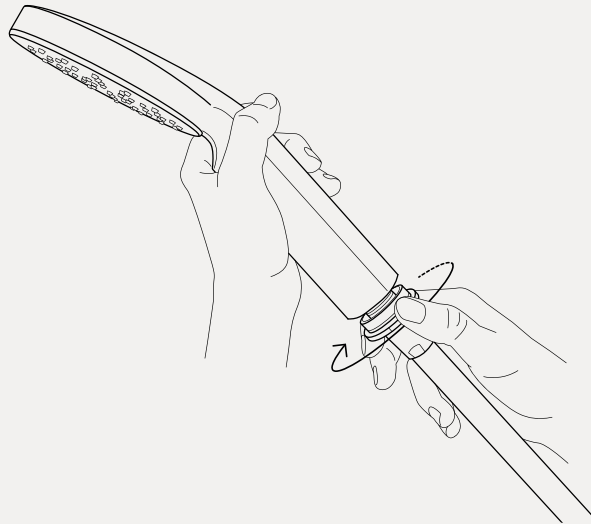


Unscrew the locking ring at the bottom of the shower filter by turning it counterclockwise by hand.

3




Remove the shower handle from the filter. Then, ensure the filter is firmly tightened to the hose so that no water leakage occurs.

4

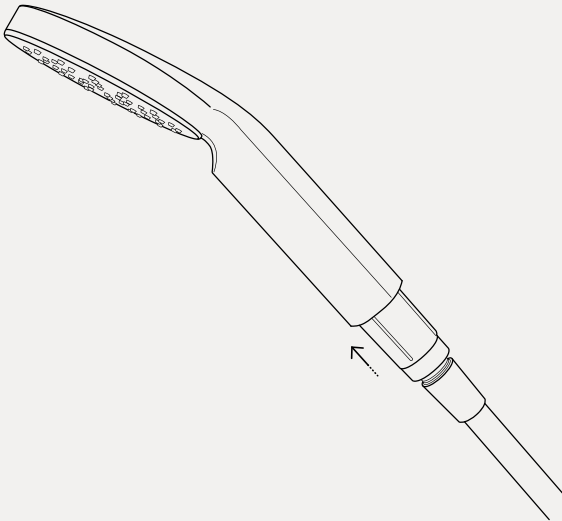

Push the filter back into the shower handle.


5

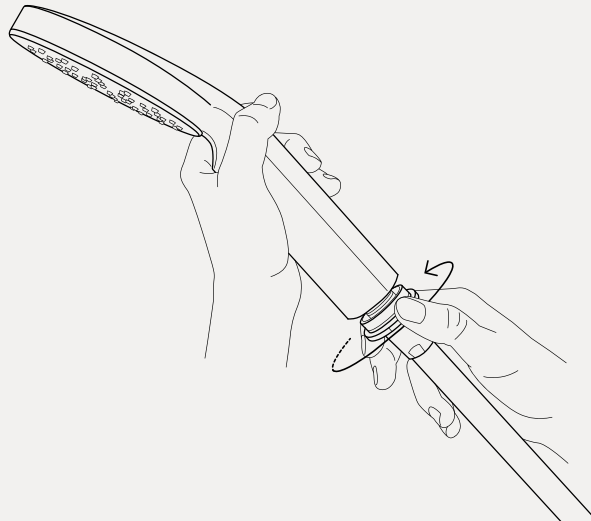

Reattach the locking ring to the shower handle by turning it clockwise to securely hold the filter in place.


Replacement of filter or installation of waterway cartridge

1

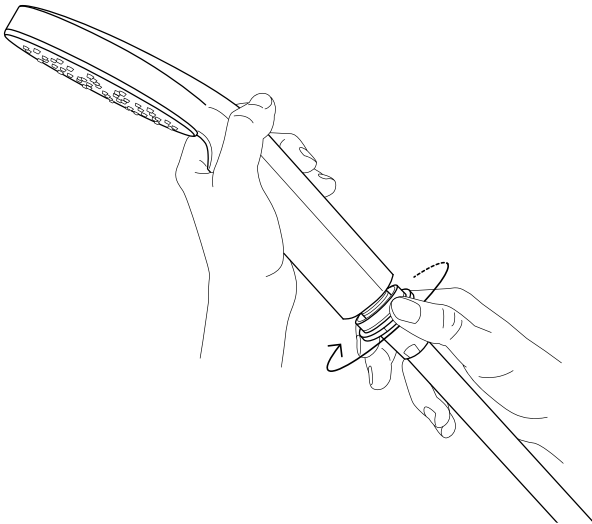


Unscrew the locking ring at the bottom of the shower filter by turning it counterclockwise by hand.

2

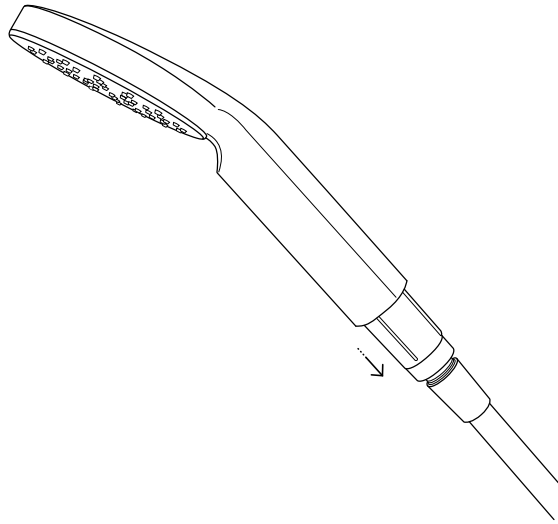


Remove the shower handle from the filter. Replace the used filter with a new one or install a waterway cartridge if unfiltered water is desired.

3

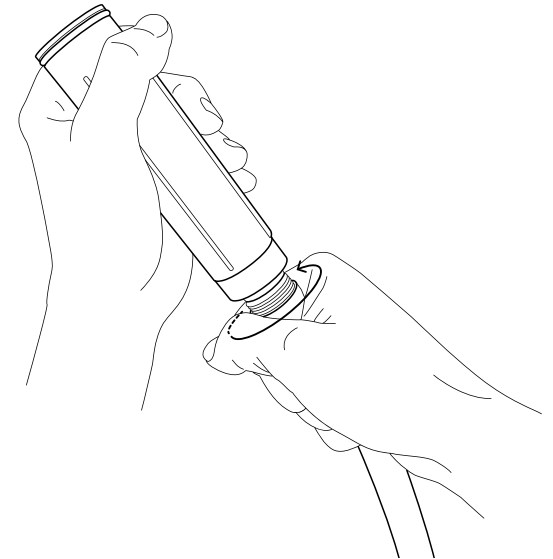


Ensure the filter is firmly tightened to the hose to prevent any water leakage.

4

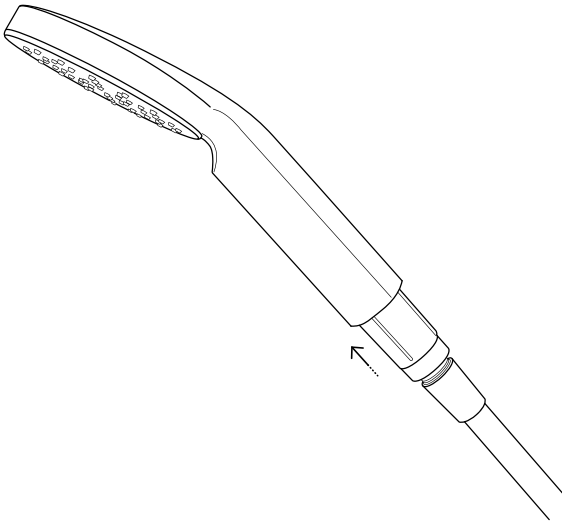

Push the filter back into the shower handle.

5

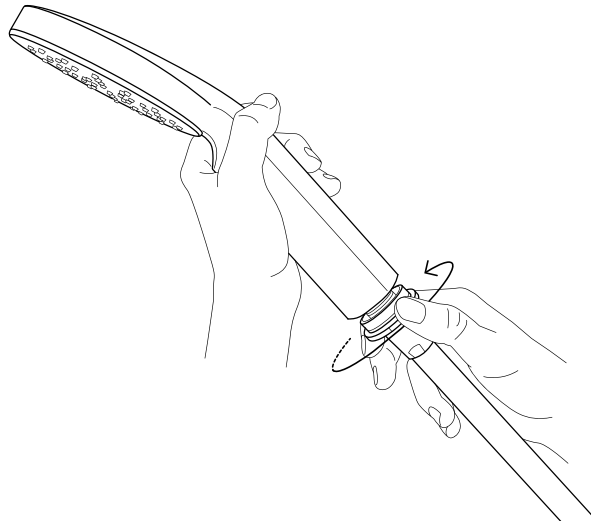

Reattach the locking ring to the shower handle by turning it clockwise to securely hold the filter in place.

Information

Surface cleaning and disinfection

If professional cleaning procedures are in place, the external surfaces of the filter may be cleaned using a new disposable wipe. Extra care should be taken to prevent contamination of any outlet of the filter. Do not disconnect the filter during cleaning.

T-safe water filters are compatible with common surface disinfectants and cleaning agents, except chlorine-based disinfectants due to their chrome plated surface. For cleaning and disinfection, the filter housing may be immersed in up to 500.000 ppm citric acid.

Storage – non-commissioned filter

The product must be stored at temperatures between 5 °C and 40 °C and at a humidity of max. 60 %. Expiry date: See label on the packaging or marking on the product.

Disposal

The filter cartridge and non-metallic parts can be disposed of with ordinary waste following applicable local regulations under the waste category number 180104 in accordance with EU Directive 75/442/EEC.

Tolerance to Water System Disinfectants

The T-safe water filters are compatible with common water system disinfectants. This has been validated by integrity testing the filters after exposure to the following disinfectants at the concentrations and contact time listed:

- 30 ppm chlorine dioxide combined with 200 mg/L suspended solids in continuous operation throughout the life cycle
- 30 ppm hydrogen peroxide in continuous operation throughout the life cycle
- 15 ppm free chlorine in continuous operation throughout the life cycle
- 10 ppm ozone in continuous operation throughout the life cycle
- 10 ppm chloramine in continuous operation throughout the life cycle
- 1000 ppm free chlorine through a 1-hour shock treatment
- 50 ppm Hypochlorous Acid (HOCL) through a 2-hour shock treatment
- 50 ppm Chlorine Dioxide through a 2-hour shock treatment
- 3500 ppm hydrogen peroxide through a 6-hour shock treatment
- 3000 ppm Silver Stabilised Hydrogen Peroxide through a 14-hour shock treatment
- An aqueous solution with pH 12 through a 1-hour shock treatment
- 1000 ppm peracetic acid at 60 °C through a 2-hour shock treatment

Precautions

- Do not use the filter if the package is open or damaged, or filter is visually damaged
- Do not disconnect the filter during the period of use
- Avoid mechanical impact and do not strike the filter with heavy or metallic objects, as this may damage the filter
- The outlet of the filter should not come into direct contact with skin or hair
- The filter must not be used continuously with water temperatures above 60 °C. The optimal working range is 10 °C – 45 °C
- The filter cannot tolerate pressure above 5 bar
- If the filter is subjected to excessively high water pressure, this will cause the filter to collapse (sudden decrease in water flow). The filter must then be discarded
- The filter cannot be “backwashed”
- The contamination level and the amount of filter-blocking particles and substances in the water might reduce the lifetime of the filter
- The filter cannot be rinsed or cleaned in a dishwasher or autoclave
- The filter must be discarded if the expiry date has passed
- Do not modify or make changes to the filter or the adapter
- Important: The filter may not be placed in a hot water bath

Note

- Do not use the water for contact lenses
- The quality of the water is not suitable for pharmaceutical use
- The filter will neither alter the taste nor the odor nor the chemical composition in the water, and does not cause any changes in the concentration of salts, metals, lime scale, chemicals, pesticides, and dyes

No liability for misprints or typographical errors.



T-safe A/S

Gydevang 1

3450 Allerød, Denmark

+45 48 17 22 82

info@t-safe.com

t-safe.com