



Instruction for Use **EN**

Easy-Fix Shower Filter VR



Description

The Easy-Fix Shower Filter VR of the T-safe Pro Water Filtration range is specifically designed to remove bacteria such as, but not limited to, *Legionella pneumophila* and *Pseudomonas aeruginosa* from tap water to prevent infections in commercial and public settings.

T-safe Easy-Fix Shower Filter VR is a complete unit equipped with a connection to fit the following RADA shower heads and panels: VR105, VR106, VR115, VR125, VR 145, PA-TF, PA-ETF, PA-15TF, PA-15TFW, PA-215TF, PA-215TFW.

The filter is supplied with a water flow regulator, which protects the filter against sudden water pressure peaks. T-safe Easy-Fix Shower Filter VR will effectively retain bacteria for a maximum period of up to 92 days, within the limitations of use as described herein.

This IFU covers the following products


Item	Code
Easy-Fix Shower Filter VR	02-092038
Replacement Filter	02-000200

Installation




To install the T-safe Easy-Fix Shower Filter VR on a Rada VR shower panel, begin by detaching the face plate by unscrewing the Allen screw.

Keep these parts in a safe place so they can be reinstalled later if necessary.

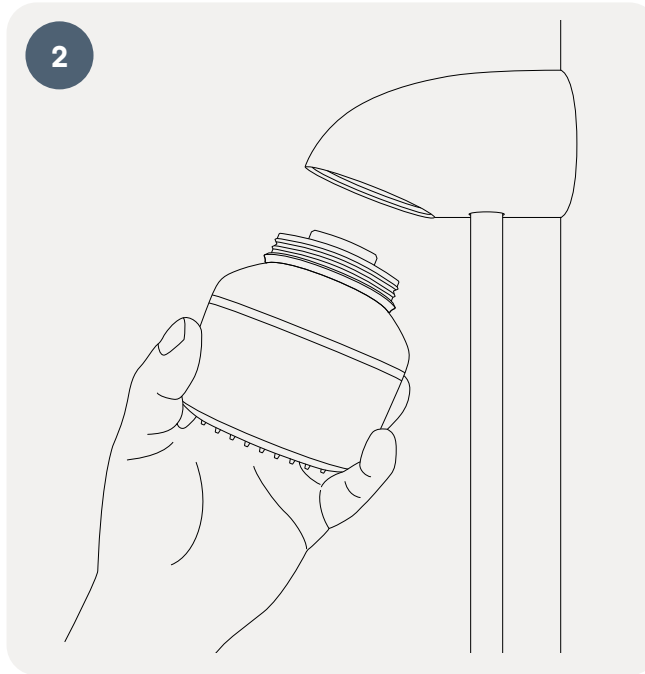


Attach the T-safe Easy-Fix Shower Filter VR to the Rada VR shower head. Securely tighten the filter housing but be careful not to overtighten.

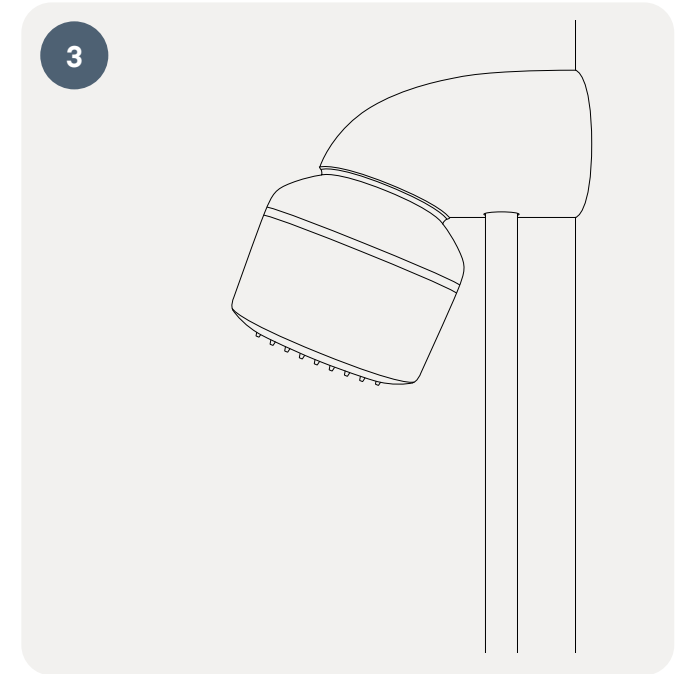


Your T-safe Easy-Fix Shower Filter VR is now ready for use.

Replacement of filter

Install the product directly from the packaging and ensure that the replacement filter does not come into contact with sources of contamination before being installed.

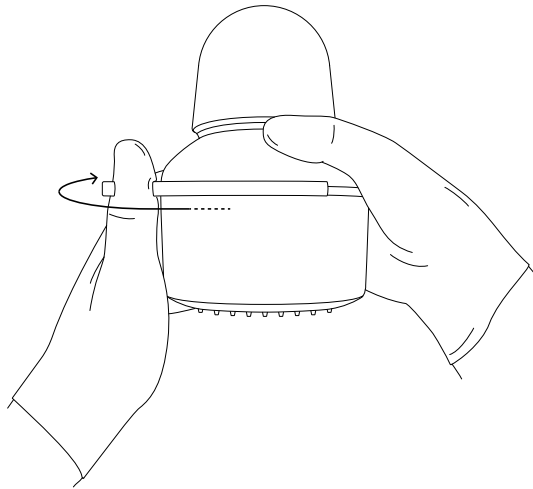
1

Remove the cover from the body.
Use the attached tool and turn the locking ring left (clockwise).

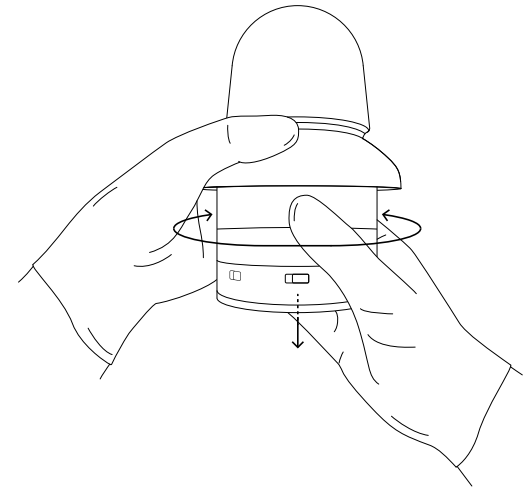2


By gently turning and pulling the filter, it is easy to remove the used filter. Make sure the used filter is disposed of correctly.

3

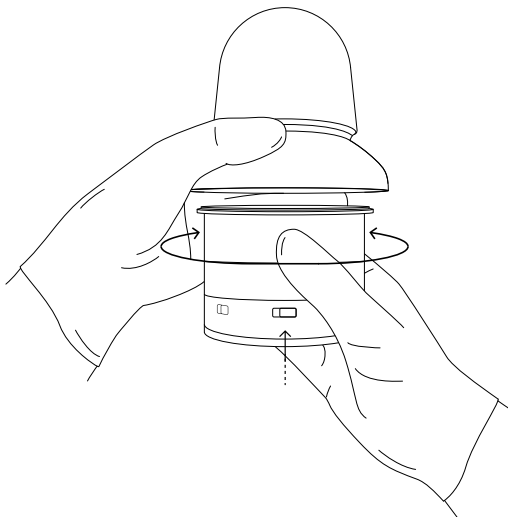

Insert the new filter by pressing and turning.

4

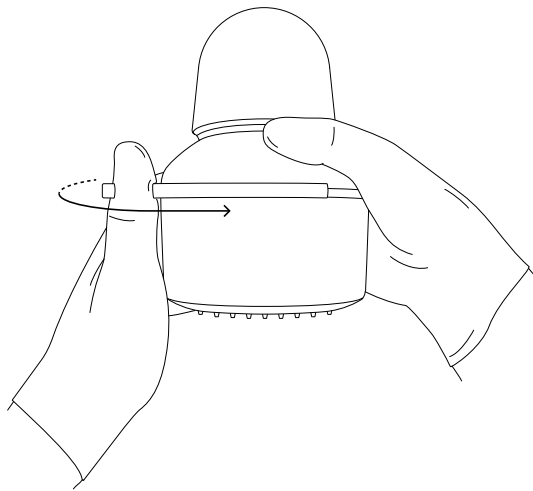

Apply the cover to the body. Fix the cover to the body by turning the locking ring counter-clockwise.

Information

Surface cleaning and disinfection

If professional cleaning procedures are in place, the external surfaces of the filter may be cleaned using a new disposable wipe. Extra care should be taken to prevent contamination of any outlet of the filter. Do not disconnect the filter during cleaning.

T-safe water filters are compatible with common surface disinfectants and cleaning agents, except chlorine-based disinfectants due to their chrome plated surface. For cleaning and disinfection, the filter housing may be immersed in up to 500.000 ppm citric acid.

Storage – non-commissioned filter

The product must be stored at temperatures between 5 °C and 40 °C and at a humidity of max. 60 %. Expiry date: See label on the packaging or marking on the product.

Disposal

The filter cartridge and non-metallic parts can be disposed of with ordinary waste following applicable local regulations under the waste category number 180104 in accordance with EU Directive 75/442/EEC.

The lead-free brass metal parts can be cleaned, descaled, and disinfected to be reused with a replacement filter cartridge. When disposing of the metal parts please ensure to recycle correctly, following local guidelines.

Tolerance to Water System Disinfectants

The T-safe water filters are compatible with common water system disinfectants. This has been validated by integrity testing the filters after exposure to the following disinfectants at the concentrations and contact time listed:

- 30 ppm chlorine dioxide combined with 200 mg/L suspended solids in continuous operation throughout the life cycle
- 30 ppm hydrogen peroxide in continuous operation throughout the life cycle
- 15 ppm free chlorine in continuous operation throughout the life cycle
- 10 ppm ozone in continuous operation throughout the life cycle
- 10 ppm chloramine in continuous operation throughout the life cycle
- 1000 ppm free chlorine through a 1-hour shock treatment
- 50 ppm Hypochlorous Acid (HOCL) through a 2-hour shock treatment
- 50 ppm Chlorine Dioxide through a 2-hour shock treatment
- 3500 ppm hydrogen peroxide through a 6-hour shock treatment
- 3000 ppm Silver Stabilised Hydrogen Peroxide through a 14-hour shock treatment
- An aqueous solution with pH 12 through a 1-hour shock treatment
- 1000 ppm peracetic acid at 60 °C through a 2-hour shock treatment

Precautions

- Do not use the filter if the package is open or damaged, or filter is visually damaged
- Do not disconnect the filter during the period of use
- Avoid mechanical impact and do not strike the filter with heavy or metallic objects, as this may damage the filter
- The outlet of the filter should not come into direct contact with skin or hair
- The filter must not be used continuously with water temperatures above 60 °C. The optimal working range is 10 °C – 45 °C
- The filter cannot tolerate pressure above 5 bar
- If the filter is subjected to excessively high water pressure, this will cause the filter to collapse (sudden decrease in water flow). The filter must then be discarded
- The filter cannot be “backwashed”
- The contamination level and the amount of filter-blocking particles and substances in the water might reduce the lifetime of the filter
- The filter cannot be rinsed or cleaned in a dishwasher or autoclave
- The filter must be discarded if the expiry date has passed
- Do not modify or make changes to the filter or the adapter
- Important: The filter may not be placed in a hot water bath

Note

- Do not use the water for contact lenses
- The quality of the water is not suitable for pharmaceutical use
- The filter will neither alter the taste nor the odor nor the chemical composition in the water, and does not cause any changes in the concentration of salts, metals, lime scale, chemicals, pesticides, and dyes

No liability for misprints or typographical errors.



T-safe A/S

Gydevang 1

3450 Allerød, Denmark

+45 48 17 22 82

info@t-safe.com

t-safe.com